



# thinka Rapid Test series

CHW



ARKRAY provides  
rapid and high accuracy  
Test Kit series

# thinka Canine Heartworm Ag Test kit CHW



Perform the Canine Heartworm test simple and quick in one step.

**Simple :** Only drop the sample.  
Easy measurement in 1 step.


**Quick :** Within 10 minutes, you can get the result.

**High accuracy :** Easy to determine the test line with high sensitivity.

## Performing a test



Drop a sample (2 drops) with pipette included.

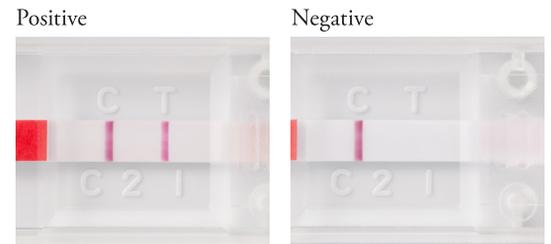

Determine the result

## Clinical evaluation

### Correlation with Other product A

		product A	
		Positive	Negative
This product	Positive	15	0
	Negative	0	17

positive concordance rate(%) =  $15 / 15 \times 100 = 100\%$

negative concordance rate(%) =  $17 / 17 \times 100 = 100\%$

### Correlation with Other product B

		product B	
		Positive	Negative
This product	Positive	15	0
	Negative	0	17

positive concordance rate(%) =  $15 / 15 \times 100 = 100\%$

negative concordance rate(%) =  $17 / 17 \times 100 = 100\%$

## Specification

Objective animals	Dog
Sample	Whole blood, blood serum, blood plasma
Measurement principle	Immunochromatography
Reaction time	5-10 minutes
Sample volume	Approx.80μL
Storage conditions	2 - 30°C
Expiration date	24 months
Package contain	Test cartridge (10 tests) Disposable Pipette (10 pcs)

## ARKRAY GLOBAL BUSINESS, INC.

Kyoto Miyuki Bldg. 10F, 689 Takanna-cho,  
Nakagyo-ku, Kyoto 604-8153, JAPAN

TEL +81-75-662-8967 FAX +81-75-662-8973